


Nitro Slow-Down Brake

Item # 10266-BRAKE ENH

Warning

Please consult your healthcare professional to determine the correct amount of brake pressure for your needs. Failure to do so could result in injury. Slow-down brakes are designed for users who are unable to safely use hand brakes. The slow down brakes allow you to manually control speed, and may also aid in gait training.

Installation Instructions

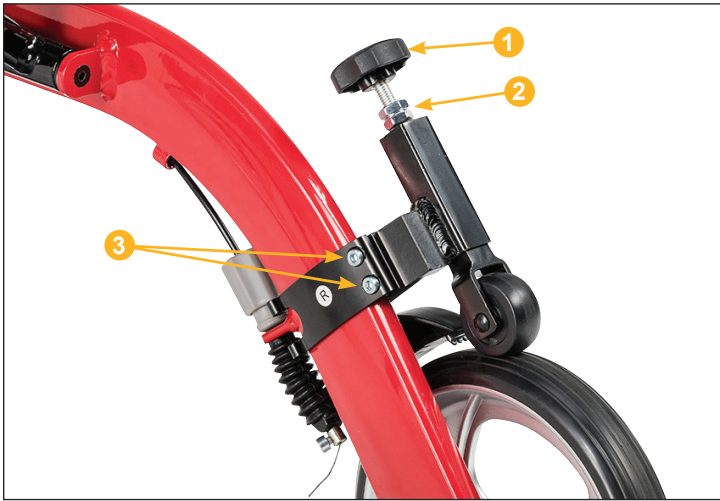

1. Ensure the adjustment knob and nut (1 and 2) are rotated to their highest position. This will allow room to apply slow-down pressure.
2. The slow-down brakes are labeled L and R. Place the L brake on the left side of the rollator frame, and the R brake on the right, from the perspective of when you walk with the rollator. The clamp for the slow-down brakes shall be placed around the brake mount weld.
3. Tighten the installation screws (3) with a size 5/32" hex key.
4. Rotate the adjustment knob (1) clockwise to apply desired slow-down pressure.
5. Tighten the adjustment nut (2) with a 1/2" wrench.