

UpWalker H200 Series Repair Instructions – REPLACING ARMREST

Replacing Armrest

Tool: 4mm hex wrench

Procedure

1. Using the 4mm hex wrench to remove the 4 screws that attach the armrest to the armrest support base from the underside. Throw the 4 screws to trash can.
2. Remove the armrest to be replaced.
3. Take out the 4 screws attached to the underside of the new armrest and set them aside.
4. Place the new armrest onto the armrest support base.
5. Insert the 4 screws coming with the new armrest into the corresponding holes on the armrest support base and screw them into the armrest with fingers so that they will stay in place.
6. Using the 4mm hex wrench to drive in the screws until they are tightly secured in place.
7. Try the armrest to ensure that it works as expected.

Estimated Time: 5 - 10 minutes for first-time repair, and 5 minutes for experienced repair.