

UpWalker H200 Series Repair Instructions – REPLACING HANDGRIPS

Replacing Handgrip (Including the upper Handgrip and the Sit-to-Stand Handgrip)

Tools: A utility knife; flat-blade screwdriver; Windex type spray glass cleaner

Procedure

1. **Method 1:** Using the utility knife slit the handgrip to be replaced along the length direction. Make sure that the cut does not fully go through the foamed handgrip to avoid damage to the surface of the aluminum tube inside. **Method 2:** Insert flat-blade screwdriver between aluminum tube handle and grip; lift screwdriver to spread apart. Spray glass cleaner inside grip. Wiggle grip to loosen and slide off handle.
2. Peel off the handgrip.
3. Remove the plastic end cap on the tube. This can be done by wiggling the cap forcefully, or by gently hitting the cap at the base on the tube side with a screwdriver and a hammer.
4. Spray some Windex into the new handgrip for lubrication. Use a piece of tissue to wipe the fluid around and apply it to the circumferences at both ends of the handgrip.
5. Slide the handgrip onto the handle tube and position as desired. This process should be easy if the handgrip is properly lubricated. Make sure that the handgrip is not twisted.
6. Put the end cap onto the tube end. This can be helped by gently tapping the cap with the hammer (or a rubber mallet) to drive it into the tube.
7. Adjust the handgrip to eliminate the gap between the handgrip and the end cap. Allow 15-20 minutes to dry before using.

Estimated Time: 5 - 10 minutes for first-time repair, and 5 minutes for experienced repair.